

Item / Art: 520-4000

Slow down brake for Maverick rollator

Ralentissez frein pour Maverick deambulateur
Freno de desaceleración para andador Maverick


1. Components

- Left side slowdown brake x 1 pc
- Right side slow down brake x 1 pc
- 5mm washer x 4 pcs
- 11.7mm washer x 2 pcs
- Axle bolt nut

2. Tools required / Outils Necessaires / Herramientas necesarias


- 6mm Hex Key
- 4mm hex key
- 13mm wrench

Step 1: Remove the rear wheel

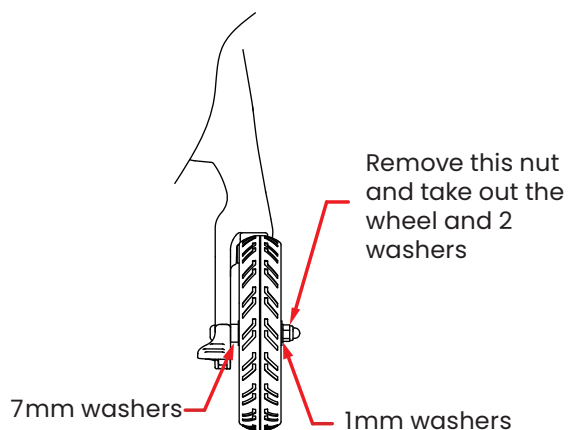


Remove washers and discard

Step 2: Remove brake pivot screw

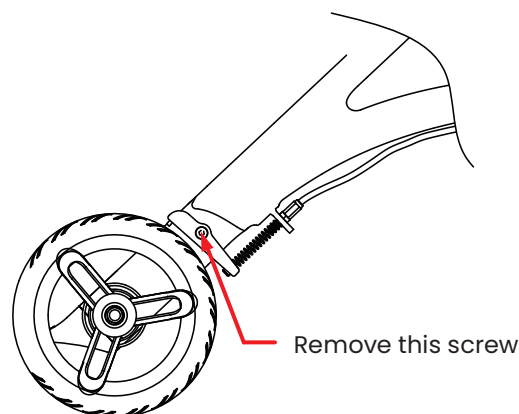


Step 3: Install slow down brake

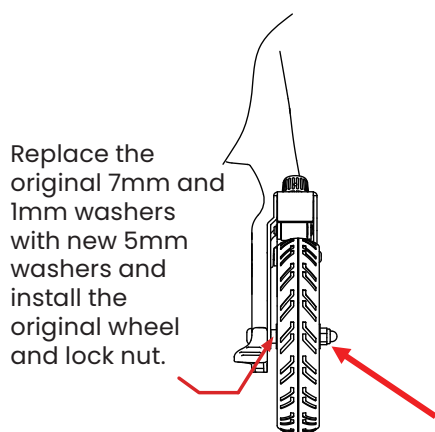

Install the new 5mm washer and nut.

Step 4: Install brake pivot screw and washer

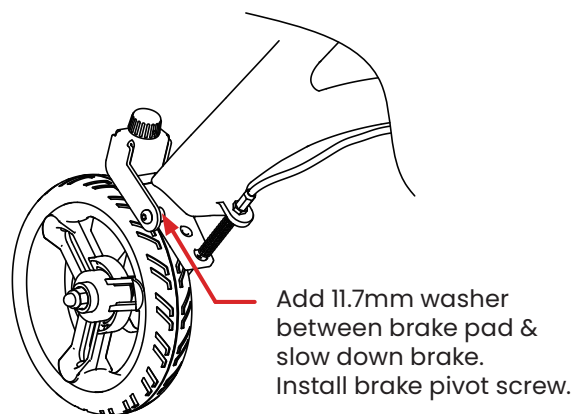


Item / Art: 520-4000

Slow down brake for Maverick rollator

Ralentissez frein pour Maverick deambulateur
Freno de desaceleración para andador Maverick

5. Use / Utilisez / Uso

Rotate knob to adjust
the slowdown brake

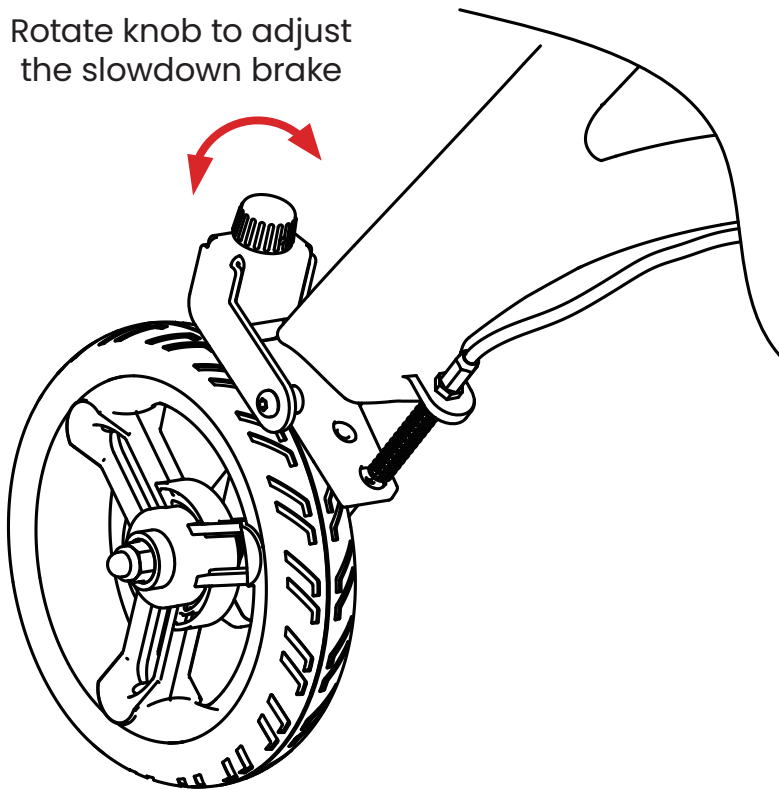

Triumph Mobility, Inc

151 Bentley Ave., Unit 4, Nepean ON K2E 6T7

Phone: 855.546.0711 | Fax: 866.225.3059

Email: info@triumphmobility.com

Web: triumphmobility.com